


DERMA SCULPT





THE ORIGINAL MICROCANNULA


The Blunt, Flexible Microcannula
For Atraumatic Filler Injections!

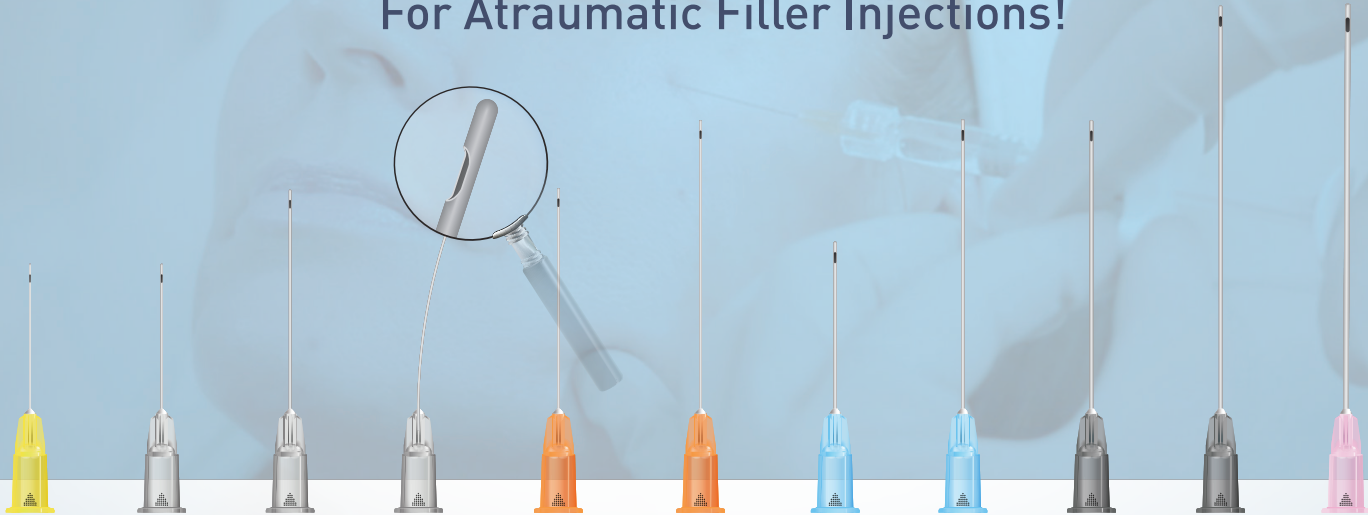

DERMA**SCULPT**
THE ORIGINAL MICROCANNULA

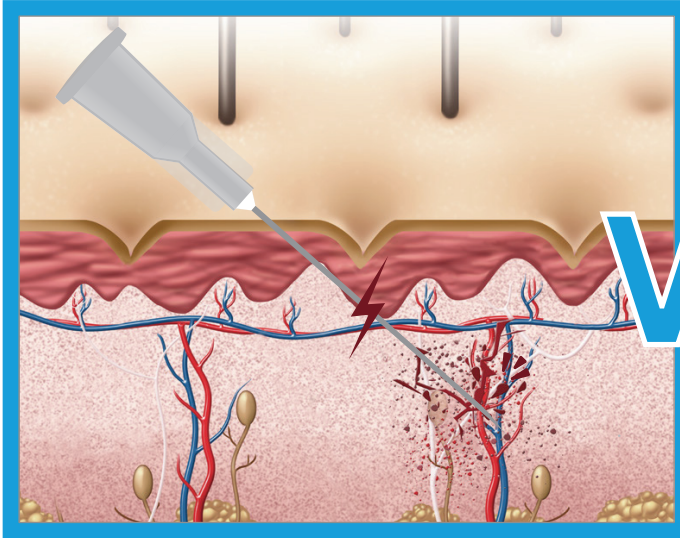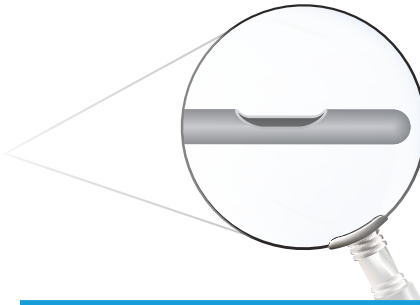

VS

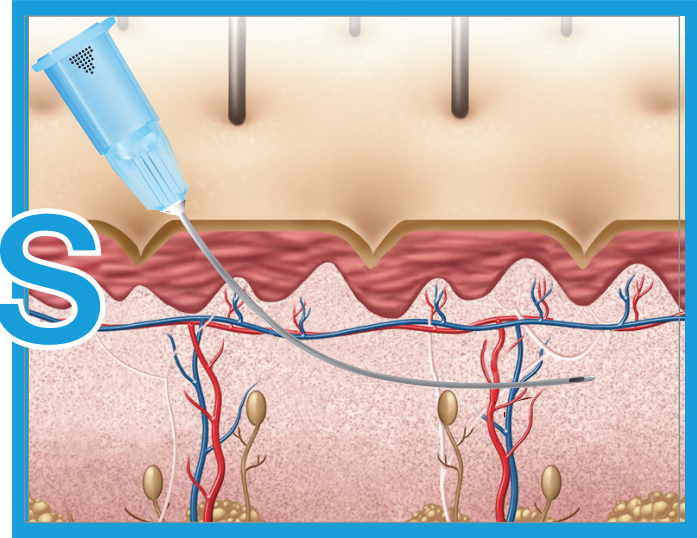

✗ Cuts through blood vessels & nerves

✓ Slides around Vessels & Nerves

COMPARED TO USING A SHARP NEEDLE

 Less chance of intravascular injection

 Virtually no bruising

Eliminating the risk of:

 Considerably less pain

 Necrosis

 Minimal downtime

 Blindness

 Single point of entry for wider coverage

 Ophthalmoplegia

 Stimulates new collagen production

**FULL
COVERAGE
WITH FEW
ENTRY
POINTS**



MAIN ADVANTAGES OF DERMASCUPT

- ✓ Much safer than using a classical sharp needle because DermaSculpt does not cut through vessels and tissues but rather navigates smoothly through the dermis and Sub-Q layer (depending on thickness and flexibility of microcannula used). The blunt end and flexibility of the microcannula allow for an almost painless and non-traumatic procedure.
- ✓ Through the fanning technique and due to the length of the microcannula, very few entry points are needed in order to treat the entire face.
- ✓ The mechanical action of the DermaSculpt microcannula on the fibroblasts generates new collagen in the skin.

THE 4 EASY STEPS OF THE SKIN SCULPTING TECHNIQUE (SST)

1 Puncture the skin using a larger-gauge sharp needle.

The gauge of the sharp needle needs to be slightly superior to the gauge of the microcannula in use.

For instance, 26G sharp needle for 27G microcannula; 24G sharp needle for 25G microcannula; etc

** These sharp needles are supplied together with the DermaSculpt Microcannulas.*

2 Insert the tip of the microcannula into the skin perpendicularly.

Insertion of the tip of the microcannula should be from a 60 to 90 degree angle until the correct depth has been found.

Pinching the skin will create a small drop of blood out of the puncture hole which should help you locate the entry point.

** Place the index finger of the injecting hand close to the tip of the microcannula to help guide the cannula into the skin. Stretch the skin opposite direction with the non-injecting hand to help enlarge the puncture hole.*

3 Slide the DermaSculpt microcannula into the skin.

As you are entering the skin, rotate the cannula tangentially to the skin surface using the flexibility of the microcannula.

If any resistance is encountered, go back and forth without forcing.

** Depending on the gauge you've selected for a specific indication or for a specific filler, the microcannula will start navigating either in the deep dermis or the Sub-Q layer.*

4 Inject filler in a retrograde fashion, fanning the entire area to be treated.

Start injecting while pulling the length of the microcannula in and out of the skin. To change direction, pull the cannula out of the skin almost entirely, back to step two (leaving the tip at the desired depth).



AVAILABLE SIZES:

For low concentration and viscosity HA fillers, superficial injections



30G x 1" (25mm)

tear trough, fine lines - precise work

For fillers of medium concentration and viscosity HA fillers



27G x 1" (25mm)

precise work



27G x 1 ½" (40mm)

most versatile size, used for most facial indications



27G x 2" (50mm):

extra length to cover larger treatment area

For medium to thick HA and non HA fillers, median to deep injections



25G x 1½" (40mm)

can be used for most facial indications



25G x 2" (50mm)

extra length to cover larger treatment area

For viscous & highly concentrated fillers, injections deep into the skin to create volume



23G x 1⅛" (30mm)

precision for thick fillers



23G x 2" (50mm)

precision for thick fillers



22G x 2" (50mm)

most versatile size for thick fillers



22G x 2¾" (70mm)


perfect for the hands

For facial fat graft (not recommended for fillers)



18G x 2¾" (70mm)

DEMONSTRATION VIDEOS ON WWW.COSMOFRANCE.NET/VIDEOS

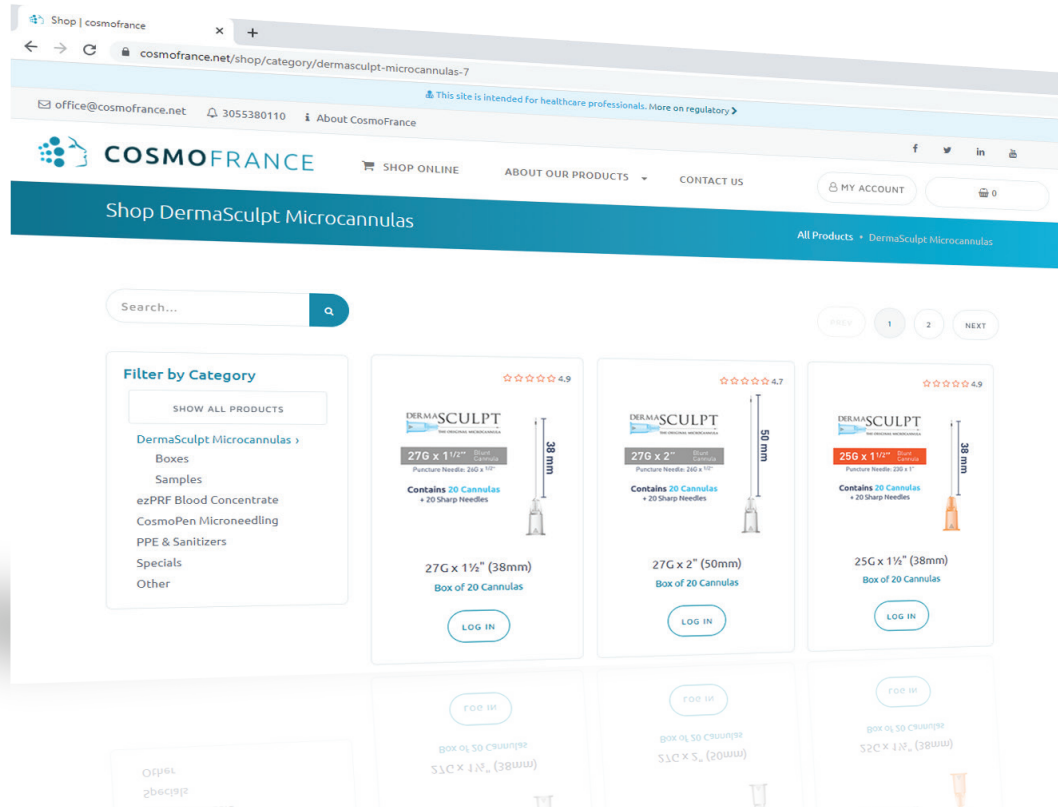



REVIEWS / TESTIMONIALS:

Rated **4.85 / 5** stars. Based on 200+ user reviews

SHAWN ALLEN, MD - Boulder, CO

The use of DermaSculpt microcannulas changed my cosmetic filler practice dramatically. Studies and experience have shown patients have less pain, redness, and swelling and there is increased precision of the placement of the filler with blunt microcannulas over sharp needles [...]

KIAN KARIMI MD - Santa Monica, CA

I used the DermaSculpt Microcannula to perform lip augmentation and I was very impressed with the fact that there was nearly no bruising after the procedure [...]

JOEL L. COHEN, MD - Denver, CO

I have found the Dermasculpt cannulas to be incredibly helpful in minimizing bruising and helping to achieve outstanding results in soft tissue augmentation of the cheeks, infra-orbital hollows, dorsal hands as well as the décolletage. Once you start using these cannulas, you totally get hooked [...]

» more reviews on www.cosmofrance.net/reviews



2121 SW 3rd Ave, Ste 500
Miami, FL 33129
☎ 1 305 538 0110 | 📠 1 305 675 2642
contact@cosmoFrance.net



Shop for DermaSculpt online
www.cosmoFrance.net

